
HEALTH KIT USER GUIDE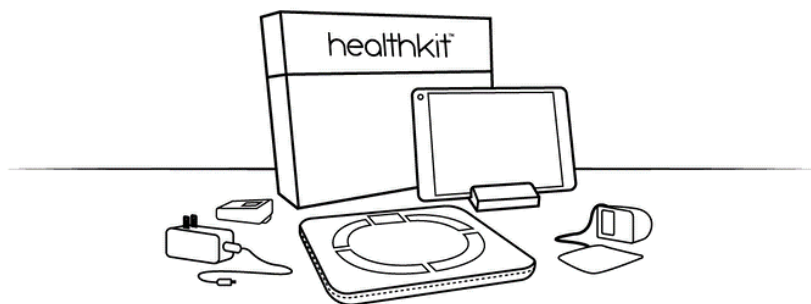

vivifyhealth™

Version 8.0

TABLE OF CONTENTS

| | |
|--|------------|
| INTRODUCTION | 1 |
| GETTING STARTED | 2-4 |
| Turning on the Tablet | 2 |
| Accepting Terms and Conditions | 2 |
| Logging In | 2 |
| Completing Health Survey Questions | 3 |
| Viewing Educational Resources | 3 |
| Understanding the Tablet | 3 |
| Requesting a Call | 4 |
| ENTERING VITALS | 5-6 |
| Weight | 5 |
| Oxygen Level and Pulse | 5 |
| Blood Pressure | 5 |
| Blood Sugar..... | 6 |
| Temperature..... | 6 |
| PACKING THE KIT..... | 6 |
| FREQUENTLY ASKED QUESTIONS..... | 7 |
| Incorrect Readings | 7 |
| Powering on Tablet | 7 |
| Reset Password | 7 |
| Viewing Health Data | 7 |
| Wireless Connectivity | 7 |

INTRODUCTION

Thank you for your participation in the Remote Monitoring Program. An overview of the program is listed below. If you have trouble completing any of these tasks, ask a family member or friend for help.

Turning on the Tablet

Follow the instructions to set up and turn on the tablet. Once the tablet is turned on, you must keep it on and plugged in until you complete the program and repack your kit.

Logging In

Create a password that you will remember. Write this password down and keep it in a safe place. You will enter it every time you are prompted to log in to the tablet.

Completing Health Survey Questions

Answer all questions about your current health honestly.

Entering Vitals

Follow the instructions on how to enter your vitals. Your remote monitoring care team will be monitoring your results and will contact you if they have any concerns.

Using Daily

Your tablet will make a noise, alerting you to check your vitals. Your remote monitoring care team will decide how often this occurs. All vitals should be taken by noon, unless you are instructed differently by your remote monitoring care team.

Repacking and Returning Your Health Kit

Your remote monitoring care team will call you when participation in the program is complete and will provide instructions on how to return the Kit. Refer to Packing the Kit instructions on page 8 for repacking your health kit.

.....

IMPORTANT: Retain the box that looks like a suitcase with foam inside for repacking. Store in a secure area.

GETTING STARTED


TURNING ON THE TABLET

1. Unpack the Kit.

Unpack the contents on a table that is near a power outlet.

2. Power up the tablet.

When the charging screen displays, press and hold down the power button firmly for 10 seconds, then let go. The power button is located directly above the green dot on the tablet screen. Then wait about 60 seconds for an introduction video to play.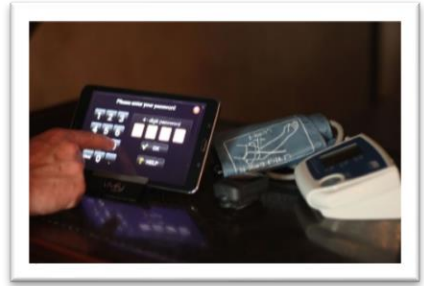

3. Place the tablet into the tablet stand.

The tablet's charging port should be on your left.

4. Plug the power adapter into a nearby outlet and insert the power cord into the tablet.

ACCEPTING TERMS AND CONDITIONS

After the introduction video has played, read and select **I AGREE** to accept the End User License Agreement (EULA). *Note: You will not be able to proceed until you've agreed to the program's terms and conditions.*

LOGGING IN

Create a password.

When the log in screen appears, create your password. Using the keypad, choose a 4-digit number that you will remember, for example your birth year (for example, '1942'). You cannot use one number four times (1111), nor can you use sequential numbers (1234). Be sure to write this number down and store it in a safe place; you will need to reference it later. When you are satisfied with your choice, select **OK** to submit your entry.

.....


IMPORTANT: This tablet is not an emergency response system. If you need emergency care, call 911 immediately.

COMPLETING HEALTH SURVEY QUESTIONS

Questions can be formatted in one of two ways:

Single Answer

Select an answer on the screen as it relates to your current health. Once you make a selection, **OK** will display to the right. If you are satisfied with your answer, select **OK** to confirm. A green check mark will appear. Proceed to the next question.

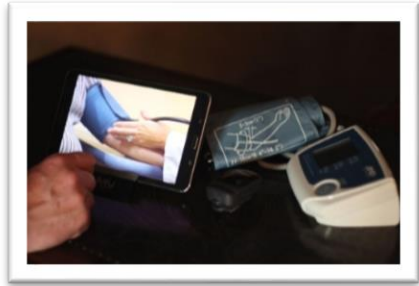Multiple Answers

Select all answers on the screen as it relates to your current health. Once you select one or more responses, **OK** will display at the bottom. If you are satisfied with your answers, select **OK** to confirm. A green check mark will appear. Proceed to the next question.

VIEWING EDUCATIONAL RESOURCES

You may be prompted to view an educational video or another screen with additional information to help you understand more about your health. Select **Now** and then **OK** to watch the video. Select **Later** and then **OK** to watch the video later. If you select later, you will not be able to go back and watch the video on your own. Your remote monitoring care team will decide when to ask you to watch the video again.

UNDERSTANDING THE TABLET

This is an image of your tablet and what it looks like when the tablet is on. You will notice a series of icons located at the bottom of the home screen. These icons represent the different tasks your care team would like you to complete. These icons will change colors throughout your participation in the program to signify the urgency of completing a task.

Following is an explanation of the icons and what the change of colors represent:

Survey Questions

Weight Scale

Pulse Oximeter

Blood Pressure

Glucometer

Thermometer

| Yellow | Gray | Red | Green Check | Orange Check |
|-----------------|------------------------|-----------------|----------------|--------------|
| Task is due now | Ready for you to start | Task is overdue | Task completed | Done for now |

REQUESTING A CALL

Note: Based on your program, the Phone icon may not be available.

To request a call with your remote monitoring care team, select the phone icon located in the bottom right corner of the home screen.

Selecting **NOW** sends an urgent request to your remote monitoring care team. They will attempt to reach you by phone or by video call on the tablet.

Selecting **LATER** sends a non-urgent request to your remote monitoring care team. In both cases, the phone icon on the main screen will turn green, indicating that you are waiting for a response from your remote monitoring care team.

To cancel the call request, select the green phone icon located on the home screen. Then select **CANCEL** (the red phone icon).

You will be sent back to the main screen, where the phone icon should now be gray. The change in color signifies that you've canceled the call, as shown in the first diagram.

IMPORTANT: If you accidentally select any of these icons, you will be prompted to complete the task at that time.

ENTERING VITALS

Based on your program, all devices may not be in your kit. With the devices in your kit, do the following:

1. **Watch the Help video.** When the tablet prompts you to record your data, select **HELP** if needed. A video will begin automatically showing how to use the device.
2. **Record your vital.** Wait a few seconds for the device to calculate your reading. A number will display on the device, then you can turn your attention to the tablet. The device should wirelessly connect with the tablet and record your data automatically. Select **OK** to accept. Select **YES** to confirm.

Manual Entry

If the device *does not* wirelessly connect and record your data automatically, select *Manual Entry*. In the fields provided, record your data. Select **OK** to accept. Select **YES** to confirm. If you realize that the number you entered is incorrect, select **NO** to clear the field and re-enter your data.

WEIGHT

It is important to weigh yourself in the morning, after you have gone to the bathroom and before you have had anything to eat or drink. Before you begin, the scale needs to be within 20-30 feet of the tablet. When the scale is not being used, it will power off automatically.

OXYGEN LEVEL AND PULSE

It is important not to wear nail polish or artificial finger nails while using the pulse oximeter. If circulation is poor, warm hands. Before you begin, the pulse oximeter needs to be within 20-30 feet of the tablet. When the pulse oximeter is out of use, it will power off automatically.

BLOOD PRESSURE

It is important to not smoke, drink caffeinated beverages, or exercise within one hour of measuring your blood pressure. In addition, if you've been instructed by your doctor to take blood pressure medication, do so one hour in advance of taking your blood pressure reading. Before you begin, the blood pressure cuff and monitor needs to be within 20-30 feet of the tablet. When the blood pressure monitor is out of use, it will power off automatically.

BLOOD SUGAR



You may have an adapter to enable your readings to be transmitted to the tablet wirelessly.

TEMPERATURE

It is recommended to use the same side ear for each reading to ensure accuracy. Body temperature will rise after a shower or exercise.

PACKING THE KIT

1. Retrieve the box your health kit came in, open and remove the user guide (which has pre-cut tape that must be used to seal the box for shipping) and remove the top and middle layers of foam.
2. Unplug the tablet from your wall outlet, power off the tablet and gather all the devices that go inside the health kit box.
3. Place the scale in the bottom of the health kit box, put the middle foam tray on top of the scale and wind up the power cord.
4. Place the power cord and adapter, pulse oximeter, blood pressure monitor, tablet stand, and tablet in their appropriate slots.
5. Remove the pre-cut tape from the health kit user guide. Place the last piece of foam on top of the tablet. Close and seal the box with the tape you just removed from the user guide. If you are unable to find the pre-cut tape, you can use packing tape instead.
6. Remove the pre-cut tape from the health kit user guide. Place the last piece of foam on top of the tablet. Close and seal the box with the tape you just removed from the user guide. If you are unable to find the pre-cut tape, you can use packing tape instead.
7. Your remote monitoring care team will provide instructions for how to return the Kit.

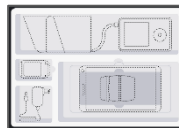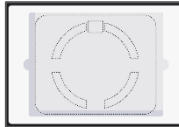FREQUENTLY ASKED QUESTIONS

INCORRECT READINGS

Why does the blood pressure monitor report elevated readings or readings that are different from another blood pressure device I've used before?

The Kit uses clinical grade devices; readings may vary from devices you already have. This is normal. Be sure to take your readings at the same time every day and while at rest. The most common cause for elevated readings is an incorrectly sized blood pressure cuff. If you believe this is the cause or have other questions or concerns about your health kit, please contact your remote monitoring care team.

POWERING ON THE TABLET

The tablet is plugged in but will not power on.

Unplug the tablet from the power cable and press and hold down the power button for 10 seconds or until the screen shows "Samsung Galaxy". Once powered on, plug the power cable back into tablet.

RESETTING THE PASSWORD

How do I reset my password?

If you forget your password, select "Help" on the login screen, then "request password reset" on the next screen. Once your request has been accepted by your remote monitoring team, your tablet will prompt you to create a new password.

VIEWING HEALTH DATA

Can I view my data once it's been logged?

Once your day-to-day vitals have been logged, you will not be able to review them. Your remote monitoring care team will be monitoring your health and will contact you should a question or concern arise.

WIRELESS CONNECTIVITY

The tablet did not wirelessly connect to the other health kit equipment to record my vitals.

Bluetooth is a short-range wireless technology that connects different devices to each other. It is important to keep the devices in your kit within 20-30 feet of the tablet. If the tablet does not receive your vital information from a device, please manually enter your readings on the tablet. It's important to complete your daily readings. Your remote monitoring team will contact you to offer technical assistance if they see excessive manual entries.

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