

vivifyhealth™

Patient Checklist

The background color will turn yellowish-red when your **Care Plan is DUE**. The tablet will ding every two minutes for five seconds.



The background color will turn red when your **Care Plan is two hours OVERDUE**. The tablet will ding every minute for five seconds.



Green checkmarks mean that no more care plans are due for today. You have completed all required tasks.



Orange checkmarks mean that another follow up entry will be due later that day, such as blood pressure or blood sugar.



To complete your care plan

Touch the screen and then tap the **Login** button.

Enter your four-digit password.

Answer all the survey questions. Use the devices to take your biometric readings.

Reminders

If you have an emergency, call 911. The tablet is not for emergencies.

Do not turn off or unplug the tablet.

Check your tablet to ensure it is plugged into wall and cord is securely plugged into the tablet.

If you forget your password, select HELP and then RESET. Your Care Team will be notified.

DO NOT throw away the health kit box your tablet and devices came in. The box will be used to package up the tablet and devices to be returned at the end of your remote care program.

Monitoring Days:



Monitoring Hours:
